



Introducing RinseDo

A unique instrument for root canal debridement

An improvement in Endodontic therapy you will appreciate

Removal of pulpal tissue, necrotic materials, and bacteria from the instrumented root canal is critical to the success of endodontic therapy. The new RinseDo handpiece is a simple-to-use, highly effective and time efficient instrument for irrigating prepared root canals. RinseDo replaces conventional manual syringe irrigation and effectively irrigates prepared root canals right to the apex. Thorough debridement of the root canal system is achieved even under difficult anatomical conditions such as small diameter or curved canals.

Gentle, safe and effective treatment

The ergonomic titanium handpiece accepts a specially designed single-use cannula with a 7mm long exit aperture. The long aperture prevents blockage of the canal and facilitates the free flow of irrigation solution. A protective stopper guards against splatter and serves as a positioning device for the saliva ejector.

During operation, RinseDo employs exclusive, patented pressure-suction technology to thoroughly flush the canal. In the pressure phase, 65 micro-liters of solution are automatically drawn from the attached syringe and aspirated into the canal. In the suction phase, the used solution is withdrawn.

The pressure generated by RinseDo irrigation is lower than the pressure created by a syringe during manual irrigation.

Simple Operation

RinseDo is powered with dental compressed air. It is operated by the footswitch of your dental unit. RinseDo quick-connects to all popular handpiece couplings. RinseDo is compatible with all irrigation solutions.



Prepared for treatment



Special cannula with protector



Cutaway drawing showing pressure-suction technology

**AIR
TECHNIQUES**
INC.



A unique instrument for root canal debridement

If you are a General Dentist performing Endodontic procedures, Endodontist, or General Dentist considering offering Root Canal Therapy then you will value Rinsendo.



Specifications

Air Consumption:	0.70 CFM
Air Supply Pressure:	35 to 60 psi
Irrigation Speed:	6.2 (ml/min)
Pulsation Rate:	1.6 Hz
Dimensions (L x W x H):	6" x 0.9" x 1.4"
Weight:	3 ounces
Rinsendo Kit includes:	Rinsendo Handpiece, Quick Connect Coupling Adapter, Disposable Cannulas (qty. 20) and Protectors (qty. 20), Seal Kit with Silicone Lubricant, Disposable Syringes 10 ml (qty. 3), Instruction Manual
Sterilization:	Rinsendo is autoclavable at 132°C.
Warranty:	Rinsendo is warranted to be free from defects in material and workmanship from the date of installation for a period of 12 months.



Rinsendo handpiece with disposable cannulas and protectors



Rinsendo fits leading handpiece couplings



Rinsendo with case and accessories

Contact your authorized Air Techniques dealer or visit www.airtechniques.com for more information.



1295 Walt Whitman Road, Melville, New York 11747-3062 ● www.airtechniques.com
516-433-7676 ● Fax: 516-433-7684 ● Toll Free in the U.S. and Canada: 1-800-AIR-TECH ● Western Facility: 954-822-2899