

## **User Instructions for Dentrrix Image Component Update 4 (CU4)**

### **What is Dentrrix Image 4.5 CU4?**

Dentrrix Image 4.5 CU4 is a software update (patch). It consists of two files, "Img45\_CU4.exe" and "DexisSensorDevice.exe." The Dentrrix Image 4.5 Component Update 4 (CU4) is installed by copying the file "Img45\_CU4.

The CU4 needs to be installed on each computer which you want to upgrade. You can use CU4 on any exe" to your local machine and executing this file (eg. Double-click the file). It is necessary to have Dentrrix 11 or higher previously installed, along with Dentrrix Image 4.5 previously installed.

Dentrrix installation from Dentrrix 11 through Dentrrix G2, including Dentrrix Enterprise 2.1 The "DexisSensorDevice.exe" file is for use by customers who want to install the DEXIS sensor and use it with their Dentrrix Image 4.5 installation. Guidelines for installing the DEXIS sensor are given below.

### **Who Should Install Dentrrix Image 4.5 CU4?**

Users who have Gendex devices utilizing the GxPicture interface who want to take advantage of the improved image quality should install CU4.

Users who want to use DEXIS hardware with their Image 4.5 software, such as the DEXcam intra-oral camera or the DEXIS PerfectSize sensor, should install CU4.

Users who have had SQL errors on closing Lab Case manager should install CU4.

Users who want to use the Air Techniques' Acclaim USB intra-oral camera with Image 4.5 should install CU4. See the instructions below for configuring this camera.

Users who want to be able to launch the Dentrrix Help files from within the Microsoft Windows Vista operating system should install CU4.

### **Configuring the Acclaim USB Intra-Oral Camera**

1. Launch Chart and select a patient.
2. Launch Image to the Digital X-ray module.
3. Select Utilities | Preferences.
4. Select the Footswitch Tab.
5. On the Footswitch/Capture Buttons dialog box, choose USB only for the new Acclaim camera.