

Acclaim USB Camera Integration for Patterson Eaglesoft and Patterson Imaging



The **Acclaim USB Camera** has two buttons on the handpiece. For these buttons to function properly an updated version of file “ESCameraService.exe” is needed. This file interprets the button commands from the camera and allows the user to freeze, unfreeze and save images all from the handpiece. The version of ESCameraService.exe needed is 14.0.0.12.

To install this file simply copy it to the location where it was originally installed, overwriting the existing version. The default installation directories for this file are:

[Patterson EagleSoft]
C:\Program Files\EagleSoft\Shared Files

[Patterson Imaging]
C:\Program Files\Patterson Imaging\Shared Files

Note: Version 13 or above of Patterson Eaglesoft or Imaging is needed.

If further assistance is needed please contact:

Patterson Technology Center

800-475-5036