

CleanStream™

Dispenser System



For Use with 1 or 2 dental units.

Directions for Use:

1. Fill the CleanStream Dispenser System with tap water to either line marked "1 L" for a 1 liter solution (1 operatory) or "2 L" for a 2 liter solution (2 operatories).
2. For a 1 liter solution of CleanStream, fill the cap of the 2.5 L CleanStream bottle to the line (represents 20ml) with CleanStream concentrate and pour into the Dispenser. For a 2 liter solution of CleanStream repeat this step.
3. Clip atomizing adapters into their nest located in cap of bottle.
4. Attach saliva ejector to smallest of atomizing adapters.
5. Attach 1 or 2 high volume ejectors to respective adapter(s).
6. With vacuum pump on and handpiece valves open, aspirate CleanStream solution.
7. For first operatory cleansing, bottle must be in upright position.
8. For second cleansing lay bottle on its flat side.
9. After each cleansing, disconnect the handpiece.

To reorder parts or entire system, refer to this Diagram:

Tips


PN ED 598



PN 57643 SMALL



PN ED 597

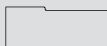

PN 57644




Cap PN 57645

O-Rings PN 57647

Bottle PN 57642

Directions for multipurpose tip usage:

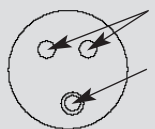


Clip tips directly into identical nests.

P/N ED 597



Use as adapter for post w/ Saliva Ejector

*Note: Use w/ 2 HVE's

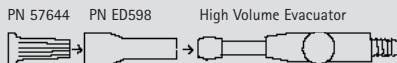

Once attached these pieces stay together.

PN ED 597



Adapter shown with Cap Overview on left.

PN 57643 Small Saliva Ejector

